

# Nilocin Wart & Verruca Treatment Pen





# For external use only Keep out of reach of children

Nilocin® contains a corrosive agent.



Read this instruction carefully prior to use

Nilorin® Wart Pen is indicated for the treatment of warts and vernicas found on the feet, hands, body arms and legs. The product can be used bu adults and children from 4 years old.









### WARTS AND VEDRUCA

Warts and verrucas (hereafter defined as warts) are local growths in the skin caused by the human papilloma virus (HPV). They may be found alone or in clusters. There exist different types of warts, most commonly found on hands, feet, elbows and knees. Warts are contagious and spread by contact of the virus with damaged skin.

# NILOCIN®

The Nilocin Wart treatment is a CE marked Medical Device. The pack comprises a treatment pen and this instruction leaflet.

The Nilocin Wart Pen contains 3 ml sufficient for minimum 220 applications Nilocin® is specifically designed for topical administration directly on warts.

Nilocin® is immediately absorbed into the wart. The neo contains a keratolytic compound, which breaks down tissue locally, and thereby eliminates warts. Nilocin® can be used for treatment of children as uoung as 4 years



# ⚠ WARNINGS & PRECAUTIONS

Only for external use. Keep out of reach of children. The product contains a corrosive agent with a strong smell. Do not inhale.

- . Do NOT use the product for genital warts or in the facial area.
- . Do not use Nilocin® if you are known to be hypersensitive or allergic to the ingredients, as well as bee/ant stings
- . When applying, avoid contact with healthy skin surrounding the wart
- · Do not expose the treated area to sunlight, until the skin has returned
- . Do not apply Nilocin® to sore or bleeding warts, irritated, inflamed. blistered or damaged skin. Pause until the skin has recovered
- Do not use on warts with hairs, birthmarks, dark moles, pigmented skin or in case you have doubts if the skin condition is a wart.
- Extra care should be taken when applying the treatment to thin and sensitive skin, and when treating children and people with dia-
- · it is better to apply too little liquid if in doubt. Over application can result in side effects such as wounds, blisters, inflammation, tissue death (necrosis), scars, and chemical burns

# · Do not exceed the number of recommended applications.

- . Consult your doctor if the treated area becomes red and painful or gets inflamed
- Single patient multiple use
- . Do not use in combination with other wart removal therapies.
- . Do not use for children under 4 years of age, unless directed by doctor. SHOULD THE PRODUCT ACCIDENTALLY COME INTO CONTACT WITH THE EYES OR MUCOSAL MEMBRANES. RINSE WITH PLENTY OF WATER FOR IS MINUTES AND SEEK MEDICAL ADVICE

### HOW TO USE NILOCIN

It is recommended to soak the wart in luke warm water for at least 5 minutes to soften the skin prior to treatment. If the wart is covered by hard skin, gently scrape the surface of the wart (e.g. with a scissors blade) and carefully remove the hard skin. Do not file the wart. It is recommended that adults apply the treatment to children.

- Remove the cap from the Nilocin pen by pressing and turning the cap counterclockwise.
- Gently massage the wart with the tip of the treatment pen for a maximum of 20 seconds. The solution penetrates immediately. Make sure to cover the whole wart and take care not to touch or spill onto surrounding skin. Recap the pen firmly after application.
- Repeat the treatment 3 times a week. a day apart. for up to 6 weeks. Do not exceed the number of recommended weekly applications

The Nilocin pen delivers a small amount of a colorless solution. If in doubt whether the tip of the pen is dry, press the tip against a piece of absorbent paper to check the liquid flow

Should any discomfort or pain occur, stop application, if it burns, sting, or if serum is applied to healthy skin, rinse with cold water.

# POSSIBLE SIDE EFFECTS

It is important to use the product as instructed. Do not overexpos as this can cause skin damage! (please see A PRICAUTIONS above).


A mild tingling or burning sensation can in some cases be felt for 10-15 minutes. If overexposed temporary side effects can occur, such as stinging redness, pain or bleeding. Pause treatment until the skin has fully recovered

There are no known risks for pregnant or breastfeeding women.

# STORAGE

Store between 2 to 25 degree Celsius and out of direct sunlight, enclosed in the original packaging and out of the reach of children. Do not use after the expiry date, or 6 months after removing the cap from the pen for the first time. Carefully recap the pen after use.

The product can be disposed with the household rubbish

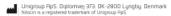



TECHNICA, DRTR - Product name: nicon Product: Wind & Venua. Treatment Pen largeridinets: Forms; acid. Drinling issorbate Rays. a.G. Englist and Drignum Valgars led Edited. Contents: 3 ms. a.G. Englist and Superitions. Registrations Name? in Scarletta and registrates of sa Marcia Devocitors to Stranger Stora et 2-25 degree (Stotas, in the original packaging, out of arest surright and out of reach and sight of children Not to be used after the eighty date. Manufacturer: Uniquo; PRS. Diptome; 373. DK-3800 Lyngby, Demmir. www.nicon.com.information.pdated acid. 2019: 07



# Nilocin Wart & Verruca Treatment Pen

# UK

### For external use only Keep out of reach of children

by adults and children from 4 years old.

Nilocin® contains a corrosive agent.



Nilnrin® Wart Pon is indicated for the treatment of warts and vernicas found on the feet, hands, body arms and legs. The product can be used













### WARTS AND VERRICA

Warts and verrucas (hereafter defined as warts) are local growths in the skin caused by the human papilloma virus (HPV). They may be found alone or in clusters. There exist different types of warts, most commonly found on hands, feet, elbows and knees. Warts are contagious and spread by contact of the virus with damaged skin.

The Nilocin Wart treatment is a CE marked Medical Device. The pack comprises a treatment pen and this instruction leaflet.

The Nilogin Wart Pen contains 3 ml. sufficient for minimum 220 applications. Nilocin® is specifically designed for topical administration directly on warts.

Nilocin® is immediately absorbed into the warf. The nen contains a keratolytic compound, which breaks down tissue locally, and thereby elir warts. Nilocin® can be used for treatment of children as young as 4 years



# WARNINGS & PRECAUTIONS

Only for external use. Keep out of reach of children. The product contains a corrosive agent with a strong smell. Do not inhale.

- Do NOT use the product for genital warts or in the facial area.
- . Do not use Nilocin® if you are known to be hypersensitive or allergic to the ingredients, as well as bee/ant stings
- . When applying, avoid contact with healthy skin surrounding the wart
- Do not expose the treated area to sunlight, until the skin has returned
- . Do not apply Nilocin® to sore or bleeding warts, irritated, inflamed. blistered or damaged skin. Pause until the skin has recovered
- Do not use on warts with hairs, birthmarks, dark moles, pigmented skin or in case you have doubts if the skin condition is a wart.
- Extra care should be taken when applying the treatment to thin and sensitive skin, and when treating children and people with dia-
- · it is better to apply too little liquid if in doubt. Over application can result in side effects such as wounds, blisters, inflammation, tissue death (necrosis), scars, and chemical burns

## . Do not exceed the number of recommended applications.

- . Consult your doctor if the treated area becomes red and painful or gets inflamed
- Single patient multiple use (1)



- . Do not use in combination with other wart removal therapies.
- . Do not use for children under 4 years of age, unless directed by doctor. SHOULD THE PRODUCT ACCIDENTALLY COME INTO CONTACT WITH THE EYES OR MUCOSAL MEMBRANES. RINSE WITH PLENTY OF WATER FOR IS MINUTES AND SEEK MEDICAL ADVICE

#### HOW TO USE NILOCIN

It is recommended to soak the wart in luke warm water for at least 5 minutes to soften the skin prior to treatment. If the wart is covered bu hard skin, gently scrape the surface of the wart (e.g. with a scissors blade) and carefully remove the hard skin. Do not file the wart. It is recommended that adults apply the treatment to children.

- Remove the cap from the Nilocin pen by pressing and turning the cap counterclockwise.
- Gently massage the wart with the tip of the treatment pen for a maximum of 20 seconds. The solution penetrates immediately. Make sure to cover the whole wart and take care not to touch or spill onto surrounding skin. Recap the pen firmly after application.
- Repeat the treatment 3 times a week, a day apart, for up to 6 weeks. Do not exceed the number of recommended weekly applications

The Nilocin pen delivers a small amount of a colorless solution. If in doubt whether the tip of the pen is dry, press the tip against a piece of absorbent

Should any discomfort or pain occur, stop application, If it burns, sting, or if serum is applied to healthy skin, rinse with cold water.

# POSSIBLE SIDE EFFECTS

It is important to use the product as instructed. Do not overexpose as this can cause skin damage! (please see A PRICAUTIONS above)

A mild tingling or huming sensation can in some cases be felt for IO-15 minutes. If overexposed temporary side effects can occur, such as stinging redness, nain or bleeding. Pause treatment until the skin has fullurecovered.

There are no known risks for pregnant or breastfeeding women.

Store between 2 to 25 degree Celsius and out of direct sunlight, enclosed in the original packaging and out of the reach of children. Do not use after the expiry date, or 6 months after removing the cap from the pen for the first time. Carefully recap the pen after use.

The product can be disposed with the household rubbish



TECHNICAL DRITH. - Product name: nicron Product. Wast & Venus. Treatment Pen largeridents: Forms and. Direthyl issorbate Roya. Carryle, and Cargarum Vulgars less Edinate. Contents. 3 mil. editors for minerum 200 popisionis. Registrations Nucleon's doctored and megaborate as a Marcial Europe. Class its Straiges Stora et 25 dispuse Stora is nicro nicro nicro popisionis. Product of a millionistic product of